

Torque Guide

Maximum implant insertion torque 50Ncm

RS
Internal Hex Connection

UF
Unified Conical Connection

RP, WP, NP, UNP
CloseFit Conical Connection

	Ncm	Cover Screw	Healing Abutment	TMA Cementing Cone
RS	15 Ncm			
UF				
RP, WP, NP				
UNP				

	Ncm	Temporary Abutment	Cement Retained Titanium Abutment	Ball Attachment	Ti-Base	Ti-Blank
RS	35 Ncm					
UF						
RP, WP, NP	25 Ncm					
UNP						

	Ncm	Straight TMA Abutment	Angled TMA Abutment
RS	35 Ncm		
UF			
RP, WP, NP	N/A		
UNP			